
Bard* Navarre* 600 mL Drainage Bag Instructions for Use _____ 1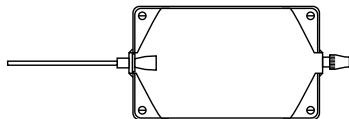

BAIRD

ACCESS SYSTEMS

Poche de drainage Bard* Navarre* 600 ml Mode d'emploi _____	2
Bard* Navarre* 600 ml Drainagebeutel Gebrauchsanweisung _____	4
Sacco per drenaggio da 600 ml Bard* Navarre* Istruzioni per l'uso _____	5
Bolsa de drenaje de 600 ml Bard* Navarre* Instrucciones de uso _____	7
Bard* Navarre* drainagezak van 600 ml Gebruiksaanwijzing _____	8
Saco de Drenagem de 600 ml Bard* Navarre* Instruções de utilização _____	10
Σάκος παροχέτευσης Bard* Navarre* 600 ml Οδηγίες Χρήσης _____	11
Bard* Navarre* 600 ml drænpose Brugsanvisning _____	13

Bard* Navarre* dræneringspåse 600 ml Bruksanvisning _____	15
Bard* Navarre* 600 ml:n drenipussi Käyttöohjeet _____	16
Bard* Navarre* 600 ml dreneringspose Bruksanvisning _____	18
Bard* Navarre* 600 ml-es folyadékgyűjtő tasak Használati utasítás _____	19
Drenážní vak Bard* Navarre* 600 ml Návod k použití _____	21
Bard* Navarre* Worek drenazowy 600 ml Instrukcja użycia _____	22
Bard* Navarre* 600 ml Drenaj Torbasi Kullanım Yönergeleri _____	24

ENGLISH

Instructions for Use:

CAUTION: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician. (Rx Only)

A. Intended Use:

The Bard* Navarre* 600 mL Drainage Bag is intended to be used for abscess, cyst, and other fluid drainage applications.

B. Device Description:

The Bard* Navarre* 600 mL Drainage Bag is for fluid drainage collection. This drainage bag can be worn around the waist, or used as a leg bag.

C. Contraindications:

None known.

D. Warnings:

1. **Before using, inspect the drainage bag for imperfections that would prevent proper function. If the bag is damaged, DO NOT USE.**
2. **After or during use, the product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practices and applicable local, state and federal laws and regulations.**
3. **Intended for single use. DO NOT REUSE. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure and/or lead to injury, illness or death of the patient.**

E. Precautions:

1. **Drainage bag must always be positioned below the drainage site to maintain drainage.**

2. **Never allow the tubing to kink. Excess tubing should be arranged to permit "straight line" drainage or coiled in a manner that does not allow the tubing to kink or fall below the level of the drainage bag.**
3. **Periodic observation of the system should be made to ensure that fluid is flowing freely and is not obstructed.**
4. **Drain or replace the bag as needed to maintain drainage collection.**
5. **Do not use after expiration date.**

F. Potential Complications:

Potential complications of percutaneous drainage procedures are site specific and may include infection.

G. Equipment Required:

The Bard* Navarre* 600 mL Drainage Bag is designed for use with Bard* Navarre* Universal Drainage Catheters and Bard* Navarre* Opti-Drain* Multi-Use Drainage Catheters. Refer to applicable Instructions for Use.

H. Directions for Use:

NOTE: The connection to the drainage catheter must be made using appropriate aseptic technique.

NOTE: The attached 24 inch (60.96 cm) line may be cut and fitted to the appropriate length. An extra luer lock is included to secure to the line if cut.

1. Remove the drainage bag from the pouch and ensure the luer connection remains sterile.
2. **Close the emptying valve on bottom side of bag by rotating clockwise.**
3. Using aseptic technique, securely attach the drainage bag inlet tube to the drainage catheter. For some procedures, it may be beneficial to place a stopcock between the drainage bag and drainage catheter. Do not over tighten the luer connection.
4. The drainage bag is attached to the patient's waist or leg or to the patient bed using the elastic/Velcro belt supplied. This bag must be suspended below the drainage site for gravity flow. Failure to do this may slow drainage and potentially increase the risk of contamination of the collection system.

5. Arrange the tubing in a straight line fashion to prevent kinking of the line.
6. Periodically observe the system to ensure adequate drainage collection. The drainage bag may be periodically exchanged or emptied, as necessary, by opening the twist open valve (rotating counter-clockwise) on the bottom of the bag.
7. Disconnect the luer connection to remove the drainage bag.

An issued or revision date and revision number for these instructions are included for the users information below. In the event two years have elapsed between this date and product use, the user should contact Bard Access Systems, Inc. to see if additional product information is available.

Revised Date: September 2010

*Bard, Navarre, and Opti-Drain are trademarks and/or registered trademarks of C. R. Bard, Inc.

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© 2010 C. R. Bard, Inc. All rights reserved.
© Copyright 2010 C. R. Bard, Inc. Med ensamrätt.
Bard Limited