English	Vacu-Cut® BIOPSY NEEDLE	Français	AIGUILLE À PONCTION-BIOPSIE Vacu-Cut®	Deutsch	Vacu-Cut® BIOPSIENADEL	Italiano	AGO PER BIOPSIE Vacu-Cut®	Español	AGUJA PARA BIOPSIA Vacu-Cut®	Nederlands	Vacu-Cut® BIOPSIENAALD	Português	AGULHA DE BIOPSIA Vacu-Cut®	Ελληνικά	ΒΕΛΟΝΑ ΒΙΟΨΙΑΣ Vacu-Cut®	Dansk	Vacu-Cut® BIOPSINÅL
<i>Read all instructions thoroughly.</i>	051768-13 185.2015-10																
Description <p>The two-part Vacu-Cut® biopsy needle consists of a biopsy cannula and an inner Trocar stylet.</p>																	
Indications <p>The Vacu-Cut® disposable biopsy needle is intended for use in obtaining histological specimens from soft tissues such as liver, kidney, prostate, spleen, lymph nodes and various soft tissue tumors.</p>																	
Contraindications <p>Good medical judgement should be exercised in considering biopsy on patients who are receiving anticoagulant therapy or who have a bleeding disorder. It is not intended for use in bone. Not suitable for use with MRI monitoring techniques.</p>																	
Storage <p>Until use the packaged product must be stored in a dry, cool and dark place.</p>																	
Note <p>– Sterilized by ethylene oxide.</p>																	
Warnings <p>– Verify that the packaging and the device are undamaged and that the sterile barrier is intact. If damaged, do not use.</p> <p>– After use, the product and accessories represent a potential biohazard. Use and dispose of them only in accordance with recognized medical practice and under observation of applicable laws and regulations.</p> <p>– This is a single use device. Reuse, resterilization, reprocessing and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.</p> <p>– This device has been designed for single use only. Reusing this medical device bears the risk of cross-patient contamination as medical devices - particularly those with long and small lumina, joints, and/or crevices between components - are difficult or impossible to clean once body fluids or tissues with potential pyrogenic or microbial contamination have had contact with the medical device for an indeterminate period of time. The residue of biological material can promote the contamination of the device with pyrogens or microorganisms which may lead to infectious complications.</p>																	
Instructions for use <p>The two-part Vacu-Cut® biopsy needle consists of a cannula with a round tip and a Trocar stylet. The nozzle of the cannula incorporates a sealing membrane on the inside of the cone which, on puncture by the stylet, forms a vacuum. Before puncture, insert the stylet into the cannula tube, pierce the sealing membrane and advance until just before the area of interest. By retracting the stylet (about 3/4 of its length) while simultaneously holding the cannula in place, a vacuum is created which will hold the excised tissue cylinder in the cannula. Press the cannula into the tissue, punching out a tissue cylinder. Tissue removal may be facilitated by repuncturing in a fanlike pattern, with the needle still inside the puncture track. While maintaining the vacuum, remove the cannula and carefully push the biopsy material obtained from the cannula into an appropriate vessel using the stylet.</p>																	

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C. R. Bard GmbH Wachhausstraße 6 76227 Karlsruhe Germany
C. R. Bard, Inc. 730 Central Avenue Murray Hill, NJ 07974 U.S.A.
Bard S.p.A. Via Cina 404 00144 Roma Italy
Bard France, S.A.S. 164-166 avenue Joseph Kessel Parklie - P14, B.P.70 78960 Voisins le Bretonneux France
Bard Limited Forest House Tilgate Forest Business Park Brighton Road, Crawley West Sussex, RH11 9BP England
Bard de España, S.A. Avenida Diagonal 652 - 656 Edificio A, 14 Planta 08034 Barcelona